

STERICAN® HANDLING INSTRUCTIONS



Carefully remove Sterican® from the packaging, ensuring not to contaminate the needle hub.



Attach the needle securely to the Luer Lock syringe by twisting the syringe clockwise while holding the protective cap close to its base.
Sterican® needles are also compatible with Luer Slip syringes.



Remove the protective cap carefully straight from the needle hub to avoid exposure to the sharp needle point.



Perform injection according to your established procedure.



Dispose the used needle immediately after application into an approved sharps container following the standard procedures of your facility and/or according to applicable regulations for sharps disposal.