


# Mini-Spike® 2

## Instructions For Use

**B | BRAUN**  
SHARING EXPERTISE

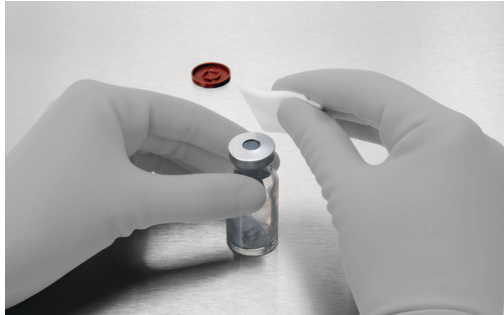

Remove protective cap from vial, disinfect stopper.



Remove the protective cap from the spike and pierce it into the vial by holding the grip plate on the Mini-Spike® 2. Make sure not to touch the female Luer lock connector on the Safeflow valve when doing so.



Disinfect the female Luer lock connector with a moistened disinfection cloth or the SwabCap® disinfection cap.



Connect to the syringe to the female Luer lock connector in the standing vial.



Flip the vial over and withdraw the solution.



To remove air from the syringe, withdraw half of the total desired quantity and flick the side of the filled syringe. Press the plunger on the syringe to expel the air from the syringe, and then withdraw the rest of the solution.



Disconnect the syringe from the female Luer lock connector on the upright vial and then withdraw the rest of the solution.



Disinfect the female Luer lock connector with a moistened disinfection cloth or the Swab-Cap® disinfection cap.

